

# **Operating Table**

## **Operating Instruction**

## 一、 Scope of Application

It is used in clinical ophthalmic surgery.

## 二、 Specifications & Main Technical Parameter

### *Dimensions of table top:*

Overall Length .....	2100mm ± 50mm
Width .....	550 ± 50mm
Height .....	600 ± 20mm
Elevation range .....	250 ± 20mm
Back panel adjustment range.....	Up fold ≥70°, Down fold ≥25°
Headboard adjustment .....	120 ± 10mm

## 三、 Structure

This product is a specialized surgical table designed for use in otolaryngology and ophthalmology surgeries. The backboard and bed body are equipped with electrically operated lifting mechanisms. The headrest features a multifunctional support arm to assist the surgeon in holding the wrist or elbow during procedures, thereby reducing fatigue. The entire unit is constructed from high-quality stainless steel, offering an aesthetically pleasing exterior that is easy to clean. It delivers stable performance, ensuring safety and reliability.

## 四、 Accessories:

1、 Arm rest: 2 Pcs

It can be mounted on either side of the table and used for intravenous drip and for measurement of blood pressure and pulse. Height and angle can be adjusted.

2、 Hand rest: 2 Pcs

It can be mounted on either side of the Head plate . Height and angle can be adjusted.

3、 Mattress on the table top: 1 Suit

It includes three parts which one connected with each other buttons. It is made up of sponge covered with imitation leather, soft and comfortable, easy to use.

## 五、 Cautions:

- ① The Head-rest adjustment is adjusted by locking screws and nuts. If the customer went adjust the head part, he must loose the corresponding handle nuts first then lock the nuts at the proper position.
- ② The patient gets in and out of the bed from the column seat, and it is strictly forbidden to get in and out of the bed from the legs.
- ③ Lubricating oil should be added regularly, so that the bed can be operated easily and flexibly, prolonging its life. Smear Vaseline over the tracks of the accessories on each side of the bed and the racks under the table, thus preventing it from being rusty.
- ④ Before putting into use, check all attachments to see if they have been locked securely so as to prevent them from loosing or sliding off while used.
- ⑤ As the bed has many parts, keep all the accessories in good order.
- ⑥ After operating every time, clean the dirt on the bed and keep the bed and accessories clean to prolong their life.
- ⑦ The ground must be kept dry. While washing the floor, please try not to make water spill into the base plate or on the casters, thus preventing them from being rusty.
- ⑧ Check the property regularly once a year.