

PPP (Plasma)gel Maker

(PPP autologous collagen transplantation)

what is PPP autologous collagen transplantation?

PPP autologous collagen transplantation is extract plasma from litigant's blood, and then made plasma liquid to gel , then inject into body.



PPP filling technology is made the plasma that extract from litigant's blood into gel, and then inject into body. During the process, our model TDD3-MC is the important equipment to made blood plasma into autologous collagen, TDD3-MC adopt advanced cooling and heat system, it can control the temperature accurately, the autologous collagen made by TDD3-MC replaced the high cost filler. With good features of no toxic and no side effect, at the same time, can extract PRP (Platelet Rich Plasma) inject into dermis, promote skin recovery, help to eliminate wrinkles, enhance the face, shrink pores , tender skin etc.

Application

1. Fill all kinds of wrinkles
2. Sculpture body
3. Fill the scar
4. Solve the cheek depression
5. Modify the chin outline
6. Fill the nose
7. Improve the profile of body part

Advantages of fill by PPP autologous collagen

1. Ideal effect: the autologous collagen can promote the cells growths, keep the skin elasticity.
2. Long time maintaining: Inject once time maintains 4-7 months.
3. Safety: its extract from body, its very safe. And
4. Low cost: Extract from own body and save much cost.

How to make PPP autologous

Step 1: Blood collection


Step2 :Blood separation

Step 3:PPP collection

Step 4:Heat PPP at 70 °C for 5 mins, then heat at 90°C for 10 mins.

Step 5: Put PPP into cooling holes for 5 mins. (set temperature at 37°C) , the the PPP gel make finished.

Step 5: Inject the PPP gel into the needed place

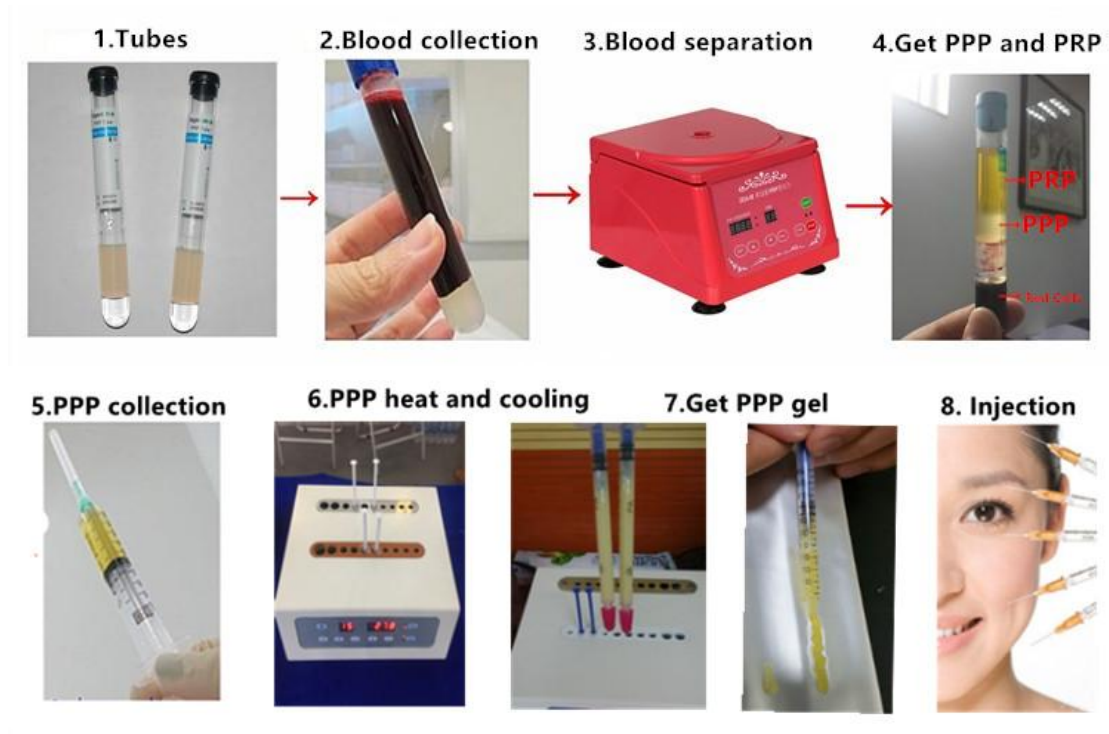

Note:

1. This machine only for PPP gel make.
2. During operation , don't need to add anything else (such as PRP).
3. Suggest don't use PRP to make Gel, because its easy to inactivity. But you could mix the PRP with PPP gel.

China Care Medical Equipment Co.,Ltd
Adress:Room B101, Zhongbang Business Building, PuXing
Road No. 87, Tianhe, Guangzhou,
Chinaweb:www.chinacaremedical.com