



Skype: chinacaremed

Facebook: China care medical

Wechat: joseph20130708

Whatsapp: +86 139 2648 0041

Email: joseph@chinacaremedical.com

Website: www.chinacaremedical.com

Add: Room B205, Changsheng Business Building, Daguan South Road No 26 ,Tianhe,Guangzhou, China

**Type1, Children Usage Bulb type,
Two Lens(MIL0, MIL1) + One Children Handle**



**Type 2, Adult Usage Bulb type,
Three Lens(MAC2, MAC3, MAC4) + One Children Handle**



Skype: chinacaremed

Facebook: China care medical

Wechat: joseph20130708

Whatsapp: +86 139 2648 0041

Email: joseph@chinacaremedical.com

Website: www.chinacaremedical.com

Add: Room B205, Changsheng Business Building, Daguan South Road No 26 ,Tianhe,Guangzhou, China



**Type 3, Adult Children Usage Bulb Type,
Five Lens (MAC2, MAC3, MAC4, MILO, MIL1) +one Handle**



Introduce,

This product is a new type of practical anesthesia laryngoscope, it is a combination of domestic and foreign similar products.

Our product adopts the ultra bright Bulb lights, the top fan space high brightness can reach 17000 MCD, it exceed all the anesthesia laryngoscope s lighting intensity, can provide clear vision for the



Skype: chinacaremed

Facebook: China care medical

Wechat: joseph20130708

Whatsapp: +86 139 2648 0041

Email: joseph@chinacaremedical.com

Website: www.chinacaremedical.com

Add: Room B205, Changsheng Business Building, Daguan South Road No 26, Tianhe, Guangzhou, China

operator.

Ultra bright Bulb is cold light source, can avoid hot bulb burn oral mucosa, The effect is better than the existing import anesthesia laryngoscope.

The handler use 3pcs AG10 button batteries, change one time batteries can used for many times operation!

The lens adopted an integration design, use more security, the lens never will be breakage.

Adopting no welding technology, making the lens environmental, more beautiful!

Size: For different Lens

MAC1, 90MM

MAC2, 100mm

MAC3, 130mm

MAC4, 155 + 5mm

MIL0, 75 + 5mm

MIL1, 102 + 2mm

Size: For different handle

28.5 mm (adult)

18.5mm (children)

Photoluminescence:

halogen bulbs (cold light sources - through fiber transmission)

Light bulb power: 2.5V600MA

Packing:

plastic box packaging. Every adult contains MAC2, MAC3 and MAC4 optical fiber lenses. One handle and one spare bulb. Every child contains MIL0, MIL1 optical fiber lens, each handle is one, spare bulb is one.

Weight: 0.7kg/ set (adult) 0.5kg/ set (children)

Single set size: 22.5cm*16.8cm*2.5cm

2, matters of attention

The user must read and understand the instructions of the product carefully before using the maintenance laryngoscope

Tracheal intubation requires professional training, and people who receive professional training can use it.

Laryngoscope may spread infectious diseases among patients, so it is necessary to sterilize and



Skype: chinacaremed

Facebook: China care medical

Wechat: joseph20130708

Whatsapp: +86 139 2648 0041

Email: joseph@chinacaremedical.com

Website: www.chinacaremedical.com

Add: Room B205, Changsheng Business Building, Daguan South Road No 26 ,Tianhe,Guangzhou, China

sterilize the two patients.

Be strictly checked before use, and do not try to use if it does not meet the requirements.

3, technical parameters

First check whether the visible tube is completely inserted into the fixed sleeve before the operation.

Check whether it is in normal working condition, for example, whether the mirror is clean, and whether there is enough brightness, such as blurring the front and rear lenses, please drop the right amount of antifogging cleaner.

The lens is hooked to the handle connector. When the lens is lifted to the operation position, the light will shine when the power is connected. After the operation is completed, the lens will be removed from the handle. Otherwise, the bulb will not extinguish, which will affect the battery life.

If it is found that the light bulb is not bright, please check whether the fiber-optic tube bulb is completely embedded in the middle of the mirror hook.

And replace the bulb, fiber tube on the base junction has a six angle screw hole, the selection of 1.5mm inner six angle to loosen the screw, pull out the fiber tube, can be poured out bulb replacement bulbs. After the fiber tube is inserted into the base hole, the locking screw, and then into the mirror hook central groove can be. Use.